Advanced Bionics AG
Laubisrütistrasse 28
8712 Stäfa, Switzerland
+41 58 928 78 00

Manufactured by:
Advanced Bionics LLC
California, U.S.A.
+1 661 362 1400

AdvancedBionics.com

029-M325-85 Rev A

© 2012 Advanced Bionics AG. All Rights Reserved.

AAA POWERPAK™ ACCESSORY CI-7403

instructions for use

English, French, Spanish, German, Dutch, Portuguese

Labeling symbols and their meanings:


	European Community Mark of Conformity. Authorized to affix the CE Mark in 20xx	
REF	Model number	 Date of manufacture
SN	Serial Number	 Manufacturer
	Type of Protection: B	 Store at temperatures between -20°C (-4°F) and +55°C (131°F)
	Fragile	 Do not get wet
LOT	Lot Number	 See Instructions for Use
	Suitable for atmospheric range between 70kPa and 106kPa, which is equivalent to 3000m above sea level to 380m below sea level.	
	Suitable for exposure to relative humidity between 0-95%	
	Dispose of in accordance with applicable national and local regulations	
IP54	The AAA PowerPak™ Accessory carries a rating of IP54. This rating indicates that the AAA PowerPak Accessory is protected against dust and splashing water.	

Warnings


- **CHOKING HAZARD:** Do not put AAA batteries in the mouth as this may pose a hazard of choking.
- Do not use leaking batteries, as this may burn the skin.
- Do not use the AAA PowerPak if the housing becomes broken, cracked or if develops sharp edges due to repeated drops.
- Do not use broken, frayed or cracked cables with the AAA PowerPak.
- Do not get the AAA PowerPak wet.
- Do not loop cables around the neck, due to risk of strangulation.
- To avoid pinched fingers while closing the latch, keep fingers clear of closing mechanism.
- To avoid an increased risk of strangulation, do not connect the Power Adapter Cables in any combination which would result in a total length (end-to-end) longer than 81 cm (32 inches).
- While no injury cases have resulted, components of the system may become hot during normal use or a fault condition. If a particular device's temperature results in discomfort or pain, promptly remove the product and contact Advanced Bionics.

AAA PowerPak™ Accessory


The AAA PowerPak Accessory is a battery powered accessory that holds 3 standard off-the-shelf disposable or rechargeable AAA batteries. The AAA PowerPak Accessory is intended to be used to provide operational power to the Auria™ Harmony™ Sound Processor or the Naída CI Sound Processor in conjunction with a processor Power Adapter, a Power Adapter Cable and a Bilateral Power Adapter (for bilateral implant recipients). This accessory is intended only for use with the Advanced Bionics Auria Harmony and Naída CI sound processors.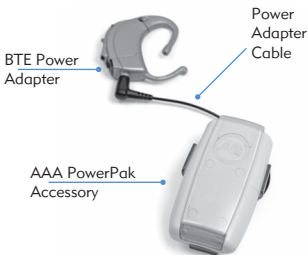

AAA PowerPak Accessory Assembled
with the Auria Harmony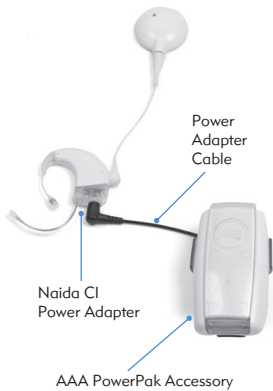

AAA PowerPak Accessory Assembled with the Naída CI sound processor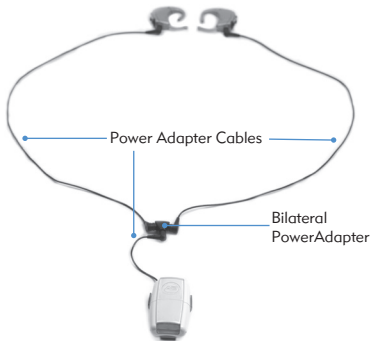

AAA PowerPak Accessory assembled
in a bilateral configuration


Assembly

Battery Insertion


Troubleshooting tip: if device does not turn on, verify correct battery orientation.

Opening and Closing the Latch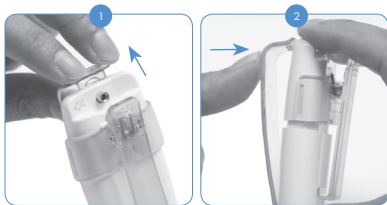


Cable Insertion to the AAA PowerPak

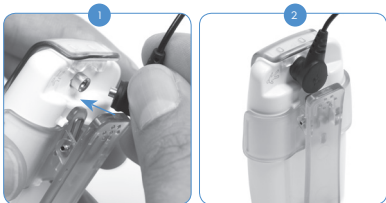Note: if a difficulty in removing the cable is encountered, it may be helpful to remove the clip or latch prior to removing the cable. Do not pull on the cable to remove.

Cable Insertion to a Processor Power Adapter

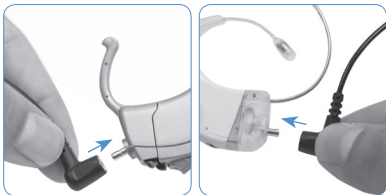Harmony
(BTE Power Adapter)

Naida CI
(Naida CI Power Adapter)

Care and Maintenance

Clean with a damp cloth. Do not use solvents.

Operating Conditions	
Temperature	0°C - 45°C
Relative Humidity	Up to 95%
Altitude	3000m above sea level and 380m below sea level

NOTE: Do not attempt to service or modify the Naída CI or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics.

External Equipment and Accessories

Model Number	Description
CI-5245-xxx	Naída CI Sound Processor*
CI-7455-100	Naída CI Power Adapter*
CI-5235-xxx	Auria Harmony Sound Processor*
CI-7404-xxx	BTE Power Adapter*
CI-7403	AAA PowerPak Accessory*
CI-7405-xxx	Power Adapter Cables*
CI-7406	Bilateral Power Adapter

*Applied part per IEC 60601-1

Note: The X's denote model number variant related to product color and/or length.

Approved Power Sources

Advanced Bionics has verified AAA PowerPak Accessory function with the power sources listed below; behavior when using power sources other than those listed here cannot be guaranteed. For optimal battery performance, do not mix batteries.

Power Source	Manufacturer	Model Number
North America		
AAA Battery	Energizer®	E92 (Alkaline)
	Energizer	L92 (Lithium)
	Duracell®	DC2400 (NiMH)
EU		
AAA Battery	Energizer	Ultra Plus LR03 (Alkaline)
	Energizer	L92-FR03 (Lithium)
	Duracell	HR03 (NiMH)

Please dispose of batteries in accordance with applicable national and local regulations.

Energizer is a registered trademark of the Eveready Battery Company, Inc.

Duracell is a registered trademark of Procter & Gamble.

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Energizer é uma marca registrada da Eveready Battery Company, Inc.